

# schülke -†



Alcohol-based, rapid disinfection of non-invasive medical devices and surfaces.

## mikrozid® AF wipes

### Our Plus

- broad efficacy within very short time
- residue-free
- ready to use

### Application areas

The product is suitable for disinfecting medical devices and other surfaces. Our classic product among the rapid disinfectant products is suitable for disinfecting near-patient surfaces with an increased risk of infection that require short exposure times.

### Microbiological efficacy

Efficacy	Concentration	Contact time
<b>bactericidal</b> EN13727, EN16615 - dirty conditions	ready-to-use	1 min.
<b>tuberculocidal</b> EN14348 - dirty conditions	ready-to-use	1 min.
<b>yeastcidal</b> EN13624, EN16615 - dirty conditions	ready-to-use	1 min.
<b>virucidal against enveloped viruses</b> EN14476 - dirty conditions	ready-to-use	30 sec.
<b>limited spectrum virucidal activity</b> EN14476 - dirty conditions	ready-to-use	30 sec.
<b>Polyoma SV40</b> in accordance with DVV (German Registered Association for Combating Viral Diseases) //RKI Guideline	ready-to-use	5 mins.

### Certificates

- VAH certificate
- IHO listed
- ÖGHMP certificate



# mikrozid® AF wipes

## Product data

### Composition:

100 g solution contains the following active substances:  
25 g Ethanol (94 %), 35 g Propan-1-ol

### Chemical-physical data

Color	colourless
Density	approx. 0,89 g/cm <sup>3</sup> / 20 °C / of the active solution
Flash point	27 °C / Method : DIN 51755 Part 1 / of the active solution
Form	Aqueous, alcohol containing solution on nonwoven
pH	Not applicable
Viscosity, dynamic	No data available

## Special advice

**Use disinfectants safely. Always read the label and product information before use.** The product has good material compatibility with metals and plastics (except acrylic glass and alcohol-sensitive paints). Surfaces that are particularly sensitive to alcohol, such as acrylic glass, must not be treated. When using alcoholic disinfectants, please observe the fire and explosion protection in accordance with the trade association regulation "Disinfection Work in Health Care". Keep away from heat and sources of ignition. Wearing protective gloves is advised. Further information is available on request. If handled properly, schülke guarantees a shelf life of 3 months after the package has been opened. For professional use only. Any serious incidents related to the product are to be reported to the manufacturer and competent authority. Not for final disinfection of semicritical and critical medical devices!

## Instructions for use

Thoroughly wipe the surface with the cloth and allow the active ingredient solution taking effect. Ensure complete wetting and maintain the surface moist during the entire exposure time. Ensure that all visible dirt is removed before disinfection. Treat only small areas. The wiping area depends on the room temperature and the structure of the surfaces to be disinfected. Pay attention to hygienic aspects when opening and threading the first cloth. Make sure to close the lid after use.

## Information for order

Item	Delivery form	Item no.
mikrozid® AF wipes Jumbo tub (220)	10/Carton	on request

These products are not available in every country. For more information please contact our local subsidiary or distributor.

## Related Products

- mikrozid® universal wipes
- mikrozid® sensitive wipes

## Environmental information

schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining out high quality standards.

## Expert opinion and information

Please visit our website for an overview of all available literature/ reports on the product: [www.schuelke.com](http://www.schuelke.com).



EMAS  
GEPRÜFTES  
UMWELTMANAGEMENT  
REG.-NR. D-150-00003



Schülke & Mayr GmbH holds a Manufacturer's Authorisation according to sect 13 para I German Drug Law and Certificates of GMP Compliance for medicinal products.

**schülke Headquarters**  
Schülke & Mayr GmbH  
Robert-Koch-Str. 2  
Norderstedt  
Germany  
Phone +49 40 52100-0  
Fax +49 40 52100-318  
[www.schuelke.com](http://www.schuelke.com)  
[info@schuelke.com](mailto:info@schuelke.com)

**schülke & Mayr (Asia)**  
10 Jalan Kilang  
#04-01/02/03  
Bukit Merah Enterprise Centre  
Singapore 159410  
Phone: +65 257 2388  
Fax: +65 257 9388  
[www.schuelke.com.sg](http://www.schuelke.com.sg)  
[mail.sg@schuelke.com](mailto:mail.sg@schuelke.com)