

schülke →



Alcohol-based rapid disinfectant for non-invasive medical products and other surfaces on basis of alcohol.

mikroqid® AF liquid

Our Plus

- ready to use
- broad efficacy within very short time
- residue free

Application areas

The product is suitable for disinfecting medical devices and other surfaces. Our classic product among the rapid disinfectant products is suitable for disinfecting near-patient surfaces with an increase risk of infection that require short exposure times.

Instructions for use

Apply undiluted to the surface, wipe and allow taking effect. Ensure complete wetting and maintain the surface moist during the entire exposure time. Ensure that all visible dirt is removed before disinfection. Maximum yield of 50ml/m². For professional use only. Any serious incidents related to the product are to be reported to the manufacturer and competent authority. Not for final disinfection of semi-critical and critical medical devices!

Microbiological efficacy

Efficacy	Concentration	Contact time
bactericidal EN13727, EN16615, in accordance with VAH Guidelines with short contact times. - dirty conditions	ready-to-use	1 min.
tuberculocidal EN14348 - dirty conditions	ready-to-use	1 min.

Efficacy	Concentration	Contact time
yeastcidal EN13727, EN16615, in accordance with VAH Guidelines with short contact times - dirty conditions	ready-to-use	1 min.
fungicidal EN13624, EN13697 - dirty conditions	ready-to-use	2 min.
virucidal against enveloped viruses in accordance with DVV (German Registered Association for Combating Viral Diseases) //RKI Guideline	ready-to-use	30 sec.
Norovirus EN14476 - dirty conditions	ready-to-use	1 min.
Polyoma SV40 in accordance with DVV (German Registered Association for Combating Viral Diseases) //RKI Guideline	ready-to-use	10 min.
Rotavirus in accordance with DVV	ready-to-use	30 sec.
Adenovirus (type 5) in accordance with DVV (German Registered Association for Combating Viral Diseases) //RKI Guideline	ready-to-use	5 min.

Certificates

- VAH certificate
- IHO listed
- ÖGHMP certificate



mikrozid® AF liquid

Product data

Composition:

100g solution contains the following active substances:
25 g Ethanol (94 %), 35 g Propan-1-ol
Labeling according to Regulation (EC) No. 648/2004: perfumes

Chemical-physical data

Color	colourless
Density	ca. 0,89 g/cm ³ / 20 °C
Flash point	27 °C / Method : DIN 51755 Part 1
Form	liquid
pH	Not applicable
Viscosity, dynamic	not determined

Special advice

The product has good material compatibility with metals and plastics (except acrylic glass and alcohol-sensitive paints). Surfaces that are particularly sensitive to alcohol, such as acrylic glass, must not be treated. When using alcoholic disinfectants, please observe the fire and explosion protection in accordance with the trade association regulation "Disinfection Work in Health Care". Keep away from heat and sources of ignition. Wearing protective gloves is advised. Further information is available on request. The shelf-life of the opened container can be found on the packaging.

Information for order

Item	Delivery form	Item no.
mikrozid® AF liquid 1 l bottle	10/Carton	on request
mikrozid® AF liquid 10 l	1/Canister	on request

These products are not available in every country. For more information please contact our local subsidiary or distributor.

Related Products

- mikrozid® AF wipes
- mikrozid® sensitive wipes

Environmental information

schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining out high quality standards.

Expert opinion and information

Please visit our website for an overview of all available literature/ reports on the product: www.schuelke.com.



Schülke & Mayr GmbH holds a Manufacturer's Authorisation according to sect 13 para I German Drug Law and Certificates of GMP Compliance for medicinal products.

schülke Headquarters
Schülke & Mayr GmbH
Robert-Koch-Str. 2
Norderstedt
Germany
Phone +49 40 52100-0
Fax +49 40 52100-318
www.schuelke.com
info@schuelke.com

schülke & Mayr (Asia)
10 Jalan Kilang
#04-01/02/03
Bukit Merah Enterprise Centre
Singapore 159410
Phone: +65 257 2388
Fax: +65 257 9388
www.schuelke.com.sg
mail.sg@schuelke.com